# MONOBLUE

## 0,25%

Instruction for use

Isotonic sterile and apyrogenic solution for vital staining of the corneal endothelium for the corneal button evaluation.

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## 0,25%

### Composition, Properties :

Purified Trypan Blue 0.25%. Isotonic buffered vehicle

- Monopotassic phosphate
  - Disodic phosphate
  - Sodium chloride
  - Water for injection
- pH  $7.25 \pm 0.25$ , osmolarity  $300 \pm 50$  mOsm/Kg H<sub>2</sub>O.

### Description :

MONOBLUE is a purified trypan blue isotonic sterile and apyrogenic solution presented in single dose syringe containing 1 ml of a 0.25% solution. The syringe is packaged in a pouch. It is externally sterile. Each box contains 5 syringes. A set of patient labels is provided to allow traceability.

### Indications :

Vital staining of the corneal endothelium for the corneal button evaluation.

### Principle of the method :

Trypan blue is an acid dye belonging to the Azo dye family. It is traditionally used as a vital dye. Trypan blue does not go through the cytoplasm barrier of the living cells and stains the poorly hydrated tissues better than the well hydrated ones. It has a greater affinity with collagenous tissues, allowing a marked color contrast between basement membranes which are rich in collagen and the other tissues (2). Trypan blue is water soluble and it is easily washed out with saline solution. In the application of corneal button evaluation, trypan blue stains the nucleus of severely damaged endothelial cells and dead cells, as well as Descemet membrane denuded of endothelial cells. Trypan blue does not stain endothelial cells with intact cell membrane, allowing the evaluation of the viability of the corneal buttons.

### Instructions for use :

Equipment and reagents :

Optical microscope with a micrometric graticule

MONOBLUE 0.25% 1ml syringe

Sterile solution of NaCl 0.9%

### Sample :

The test is conducted on fresh or preserved cornea excised from the donor.

### Procedure :

Open the pouch inside the sterile area to withdraw the syringe. Mount an appropriate cannula. The excised cornea is laid onto a sterile dressing, endothelial side upwards. Slowly push the plunger to lay the dye into the endothelial cavity, leave in contact for 1 minute. Thoroughly and atraumatically rinse the excess dye with the saline solution. The cornea is then dipped into a saline solution bath for 4 minutes. Lay the cornea endothelial side upwards onto a sterile slide and examine under optical microscope. Using the micrometric graticule, calculate the ratio of dead cells and the endothelial cells density in the central zone and in 4 peripheral zones randomly chosen. Count the endothelial cells inside of the squares as well as the cells cut by the right or the top border of the square. The mean calculated in 1 mm<sup>2</sup> represents the density (1) The test must be as short as possible to avoid possible traumatism, these operations must be conducted in a controlled environment using sterile equipment and supplies. The evaluation of the corneal button viability must be performed by trained personnel. The left over product must be discarded as any hospital laboratory waste.

### Precautions :

Trypan blue solution is toxic, it is harmful in case of ingestion, inhalation or contact with skin or eye, it provokes irritation. Trypan blue solution is carcinogenic in case of chronic exposure and may cause fetal congenital abnormalities. Refer to the safety data sheet posted on ARCADOPHTA web site : [www.arcadophta.com](http://www.arcadophta.com)

### Single use medical device :

Do not re-sterilize.

The integrity of the device is guaranteed if the packaging is not damaged.

Do not use if the pouch is compromised.

Do not use after expiration date.

### Storage conditions :

Store in the original packaging, at ambient temperature, protected from light, in a clean dry place.

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