MONOBLUE

0,055%

Instruction for use

Sterile and apyrogenic
trypan blue solution.

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0,055%

Composition :

Purified Trypan blue / isotonic buffered vehicle

Description :

MONOBLUE is a purified trypan blue isotonic sterile solution presented in single dose syringe containing 0.75 ml of a 0.055% solution. The syringe is packaged in a pouch. It is externally sterile to allow its use in the operating theatre. Each box contains 5 syringes. A set of patient labels is provided to allow traceability.

Indications :

Temporary staining of intrastromal tissues.
Temporary staining of the Descemet membrane.
Aqueous humor temporary staining.
Crystalline masses temporary staining

Contra indications :

MONOBLUE is well tolerated by the ocular tissues. However adverse effects have been observed in the animal when Trypan blue is put in contact with retina (*). Therefore it is not advisable to use Monoblue to stain retinal tissues.
Do not use in patients with hypersensitivity history.
Do not use in pregnant women, or if there is presumption of pregnancy. Do not use in children.

Precautions :

The intraocular tissues integrity should be taken in consideration to control the staining of the target tissues and to avoid non expected staining of other tissues. Use MONOBLUE with caution in presence of hydrophilic acrylic intraocular lenses, the hydrated material having shown potential reversible absorption of the dye. Thoroughly rinse the excess product with a saline solution immediately after application.
Single use medical device.
Do not re-sterilize.
The integrity of the device is guaranteed if the packaging is not damaged.
Do not use if the pouch is compromised.
Do not use after expiration date.
Only use cannula between 20G and 24G.
Verify the good mobility of the plunger before use.

Storage conditions :

Store in the original packaging, at ambient temperature, protected from light, in a clean dry place.
Do not refrigerate.
Do not freeze.

Instructions for use :

Open the pouch over the sterile operating field or inside the sterile area to withdraw the syringe.
Mount an appropriate cannula.
Slowly push the plunger to lay the dye directly on the tissue to be stained. Thoroughly rinse the excess dye with a saline solution.
The left over product must be discarded as any hospital waste.

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