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ARCALINE : Perfluorodecalin aid for retinal surgery. Composition : Purified perfluorodecalin, sterile and apyrogenic. Product description : ARCALINE is presented in a glass vial sealed with a polymer stopper and a metal crimp cap. A pouch bag guarantees the external sterility ; the product is presented with a single use needle and syringe. 3 volumes are available (3ml ; 5 ml ; 7ml) depending on the surgical procedure. Indications : Per-operative device for retinal surgery ; in particular retinal detachment (RD), RD with giant tear, RD with proliferative vitreal retinopathy (PVR) including diabetic proliferative retinopathy, retrieval of luxated natural or artificial lens in the vitreous. Contraindications : The product should not be allowed to infiltrate under the sub-retinal tissues since there are no specific studies on tolerance. Warning : ARCALINE is a per-operative device for vitreo-retinal surgery. It should not be left in place in the eye at the end of the procedure. Care should be taken to maintain the product inside of the bubble during the injection and just in front of the retinal tear. This will prevent the product from dispersing and infiltrating the sub-retinal tissues. The volume to inject should be adapted to the procedure. It is strongly recommended not to use a glass syringe. Do not re-sterilize. Do not use after the expiration date. Do not use the needle for intra-ocular injection. SINGLE USE ONLY. Instructions for use : Fit the needle onto the syringe and pierce the stopper after having removed the metal cap. Transfer the product from the vial into the syringe. Replace the needle with a cannula for intra-vitreous injection. Use a fluid or gas exchange procedure to remove the product. Strictly reserved for surgeons trained in retinal surgery. No particular storage requirement. Store at room temperature. **STERILIZATION METHOD : Sterilizing filtration and autoclave.**

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